OSD U

4 Level

Opioid-Substitutions- therapeutika in Urin

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Opioid Substitution Drugs in Urine

Description

Medidrug® OSD U is a lyophilized reference material, prepared from human urine.

Intended use

The reference material is used for quality assurance of the determination of opioids / opioid substitution drugs in urine.

Transport and Storage

The reference material can be shipped at ambient temperature. Upon receipt the material has to be stored at +2 to +8°C.

Stability

Originally closed: +2 to +8°C, 36 months, but no longer than the expiration date specified in the certificate of analysis.

After reconstitution: +2 to +8°C, 7 days

EDDP and 6-MAM have to be analysed on the day of reconstitution!

Preparation

Reconstitute each vial exactly with 2.0 ml of HPLC-water or bi-distilled water. Replace the stopper, swirl gently and allow the reference material to stand for 30 minutes at room temperature, then gently invert the vial for 15 minutes (rotation mixer) to ensure homogeneity.

After reconstitution the reference material should be treated the same as patient specimens and run in accordance with the instructions accompanying the instrument, kit or reagent being used. Before sampling, allow the reference material to sit at room temperature and gently invert the vial for 10 minutes (rotation mixer) to ensure homogeneity.

Precautions

No test method can guarantee complete assurance that products containing human source materials will be absent of these or other infectious agents. Therefore, handle them with the same precautions used with patient specimens.

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